

A woman with long dark hair, wearing a bright red hooded jacket, is smiling and looking upwards and to the right. She is standing in a scenic mountain landscape with snow-capped peaks and a valley below. The sky is overcast with soft clouds.

Changer un système

2 pièces

Matériel nécessaire



Retirer l'appareillage usagé



Nettoyer et sécher la peau



Inspecter la stomie et l'aspect de la peau




Mesurer la stomie



Reporter le contour de la stomie sur le support



Découper le support

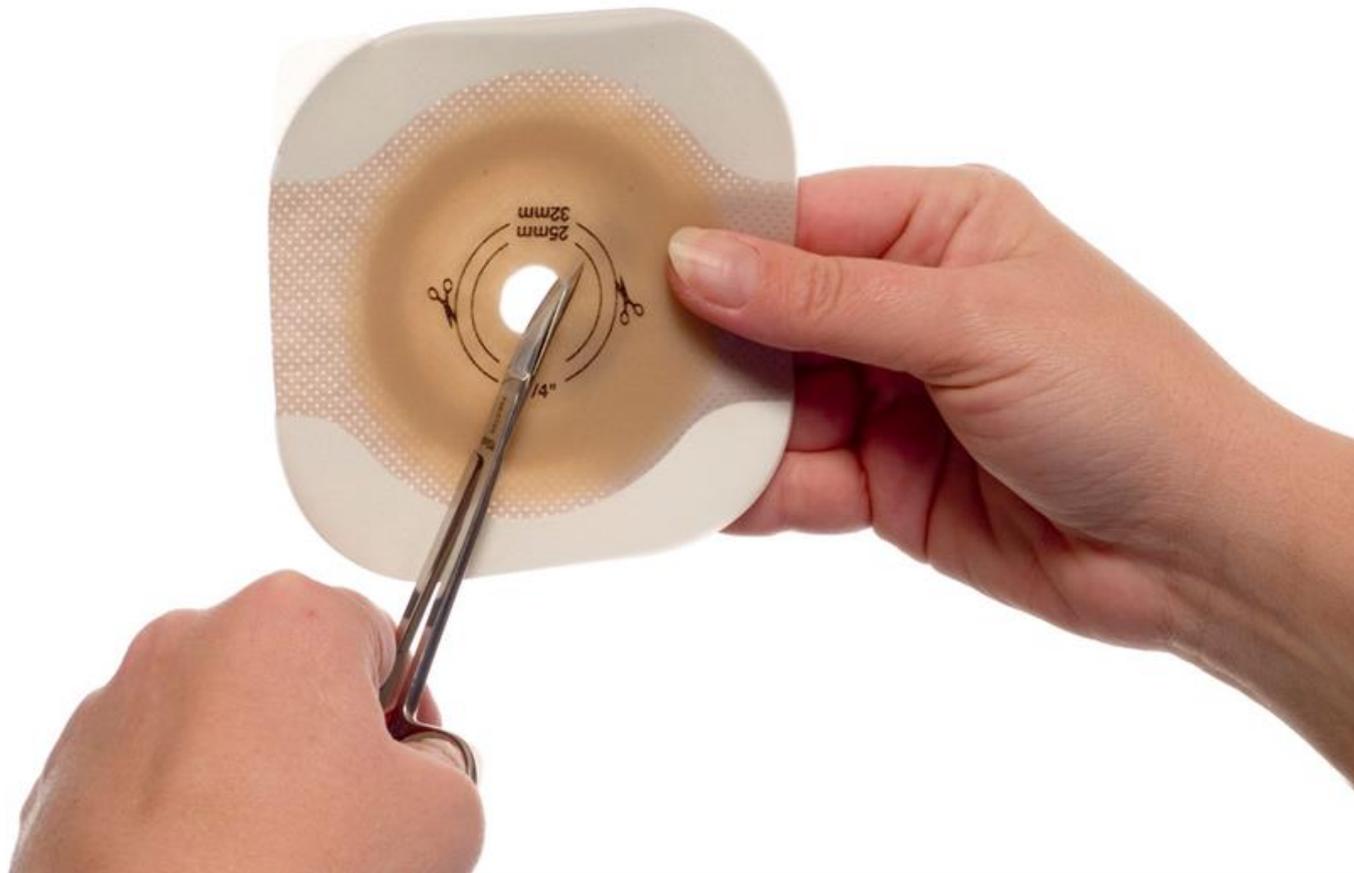

Retirer le film de protection



Fermer la poche




Robinet de vidange
d'une poche d'urostomie



Clamp
d'une poche vidable

Appliquer de la pâte ou un anneau protecteur si nécessaire

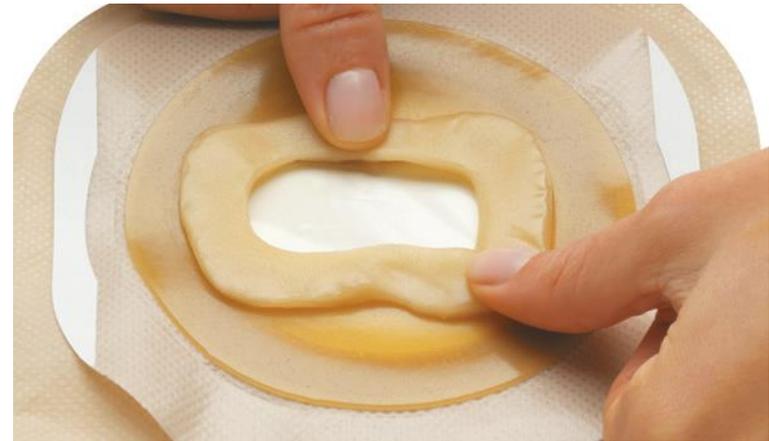

Mettre en place le support



Coupler la poche sur le support



Merci

Ce document est destiné à l'usage des professionnels de santé. Moderma Flex, Conform 2, Adapt sont des dispositifs médicaux de stomathérapie de Classe I fabriqués par Hollister Incorporated. Attention, lire attentivement la notice d'instructions et/ou les étiquetages. Ces dispositifs sont pris en charge par l'assurance maladie. Hollister et son logo, Moderma Flex, Conform 2, Adapt sont des marques déposées de Hollister Incorporated, USA. ©2020 Hollister Incorporated. RCS Nanterre 402 202 733. Édition : Octobre 2020

